

Collection Instructions



1
Go to accessmedlab.com/login and register Lab ID Card using the proper credentials under "Physician Login".



2
Place Lab ID Card or requisition form inside the specimen bag.



3
Wash hands and place all items on a clean counter top.



4
Open the clear transport tube and fill up to the 1mL line.



5
Tightly fasten the cap back onto the clear transport tube and then place the tube into the biohazard specimen bag.



6
Place sample bag inside of the kit.



7
Place kit inside the UPS Return bag and seal by peeling the adhesive strip.



8
Drop at the nearest UPS store or drop box **on the same day as collection.**